

# Rezenerate Nano Facial Treatment

## Normal Skin Brightening Protocol

### FEATURES OF TREATMENT:

Incredible Enzymes, Rare C serum along with a Rezenerate NanoFacial is a powerhouse treatment against lusterless and dull skin. It optimizes the skin's complexion on the surface and at the cellular level using a blend of botanical exfoliating enzymes and refinement assistants that work together to rejuvenate and repair tone and texture while brightening and clarifying the skin (pH 5.6). Rezenerate acts as an infusion catalyst, assisting the products and serums you are already using to treat various skin issues. The combination of Incredible Enzymes, Rare C with a Rezenerate NanoFacial will brighten, tighten and revitalize the skin.

The NanoCleansers are gentle enough for sensitive, delicate or dry skin. Sonic cleansing brush increases circulation and effectively cleanse entire face, especially effective for congested skin, it removes dirt, oil, dead skin cells and makeup residue. Rezenerate Wand Pro benefits are endless and the WandPro acts as an infusion catalyst, assisting the products and serums you are already using to treat various skin issues. The NanoGlobes help to eliminate redness, oxygenate skin, stimulate nerves, reduce pores, and sooth the skin.

### BENEFITS OF TREATMENT:

- Optimizes complexion on surface and cellular levels.
- Rejuvenates, exfoliates, and clarifies texture.
- Naturally lightens and evens skin tone.
- Diminishes the appearance of fine lines and other signs of aging skin
- Helps heal blemishes or other skin imperfections

### PRODUCTS:

- Quiet Wash
- Incredible Enzymes
- Mist Me Spritz
- Hush Hydrate Mask
- Relief Bio-Powder
- Rare C Serum
- Hush Hydrate Gel or Vital Lipid Lotion or Saffron Meristem Cream
- Broad Spectrum SPF 30

### TOOLS:

- Rezenerate WandPro
- Rezenerate Pro Nano Tip
- Rezenerate NanoCleanser Brushes
- Rezenerate NanoGlobes (*frozen or on ice*)

### STEPS FOR TREATMENT

**PRE-CLEANSE:** Apply **Quiet Wash** and emulsify with tepid water. Rinse thoroughly to remove oils, dirt and makeup from skin.

**CLEANSE:** Apply **Quiet Wash** and emulsify with tepid water. Incorporate the **NanoCleanser Brushes** to further help deep clean and detoxify the skin. Rinse thoroughly with a cool towel.

**EXFOLIATE:** Apply a thin layer of **Incredible Enzymes** to face and neck with a fan brush and massage for 3-5 minutes and rinse well. Remove with tepid towel or 4 x 4. (If necessary, perform an additional cleanse using **Quiet Wash**.)

*See page 2 for continued Steps for Treatment, Notes & Contraindications*

**STEPS FOR TREATMENT (CONTINUED)**

**TREATMENT:** Use **Rare C Serum** as your treatment Serum, apply liberally all over face with fan brush. Using **HyRez Glide** for slip, use the **WandPro** with a fresh Tip to perform the infusion portion of the treatment. Set WandPro speed according to client's tolerance. (See pages 8-10 of the Rezenerate NanoFacial Manual for detailed operating instructions and NanoFacial Sector Maps.)

**MASK:** Apply **Hush Hydrate Mask** with 2-3 shakes of **Relief Bio-Powder** liberally for 5-10 minutes. Using the previously frozen or iced **NanoGlobes**, perform the NanoGlobe Zen Acupressure Massage over the mask. (See page 12 of the Rezenerate NanoFacial Manual for Acupressure Instructions and Map.) Remove residual mask with tepid towel.

**NOURISH:** Wipe skin with wet, soft gauze or cotton soaked in **Mist Me Spritz**.

**HYDRATE:** Apply choice of moisturizer (**Hush Hydrate Gel**, **Vital Lipid Lotion** or **Saffron Meristem Cream**) to face and neck.

**PROTECT:** Apply **Broad Spectrum SPF 30** to face and neck.

**NOTES AND RECOMMENDATIONS:**

- Prior to using this product, it is recommended to patch test all clients especially with sensitive and Health-Challenged Skin®.
- Wear gloves and sanitize regularly.
- Use steam & hot towels with extreme caution. Have cool towels available if client is agitated by ingredient activity to remove product quickly.
- Do not heavily rub or try to penetrate the product into the skin.
- Check the client's health history for medications, allergies or sensitivities to any ingredients in this product.
- Do not leave product on skin longer than the recommended time.
- Follow all protocols and instructions for the Rezenerate WandPro, Pro Nano Tips, NanoCleansers, and NanoGlobes listed in the Rezenerate NanoFacial Manual. (Manual located in your RAP Online Portal, and Online Education Gateway.)

**CAUTIONS AND CONTRAINDICATIONS:**

- Do not use on skin with open lesions, sores or wounds.
- Do not use on broken/cracked, sun-burned or rashy skin.
- Do not use on open comedones, current breakouts or cystic acne (grades 3 & 4).
- Do not use on clients with compromised barriers or active skin conditions.
- Do not use on clients who have allergies to natural fragrances.
- This product contains four enzymes (pineapple, papaya, fig and pomegranate). The use of SPF is mandatory for 2 weeks daily following application of this product.
- It is recommended that use of Vitamin A (retinals and retinols), enzyme masks or scrub/polishing products not be used within 5 days of this treatment.
- Do not wax or sugar 3 days before or one week after treatment.
- Check with physician if client is pregnant, breast feeding, on chemotherapy drugs, have a severe autoimmune disease or any health condition that may need a physician's release to perform this treatment.
- Avoid contact with eyes. Keep out of reach of children.
- Please refer to a Physicians Desk reference manual to assure there will be no conflicts with prescribed or OTC medications and ingredients used in this product.
- If any reactions occur during treatment, cease the use of this product immediately.
- Avoid all contraindications for Rezenerate WandPro, Pro Nano Tips, NanoCleansers, and NanoGlobes listed in the Rezenerate NanoFacial Manual. (Manual located in your RAP Online Portal, and Online Education Gateway.)

\*Please refer to the specific product data sheets with additional questions, concerns and for the complete ingredient listing for each product.

\*Please refer to the Rezenerate NanoFacial Manual for all questions regarding the Rezenerate WandPro, Pro Nano Tips, NanoCleansers, and NanoGlobes. Email Rezenerate clientrelations@rezenerate.com with any questions or concerns.