

# Rezenerate Nano Facial Treatment

## Sensitive Skin Protocol

### FEATURES OF TREATMENT:

This treatment is for those clients with sensitive or sensitized skin. This restorative treatment specifically reduces inflammation, redness, and itchiness while calming, healing and nourishing the skin. This treatment also adds a great exfoliation and deep hydration that can improve complexion and boost your skin's immunity which will help prevent future irritations. The combination of Refine Polish, Soothe Essence Serum with a Rezenerate treatment will brighten, calm and nourish sensitive skin.

The NanoCleansers are gentle enough for sensitive, delicate or dry skin. Sonic cleansing brush increases circulation and effectively cleanses entire face, especially effective for congested skin, it removes dirt, oil, dead skin cells and makeup residue. Rezenerate WandPro benefits are endless and the WandPro acts as an infusion catalyst, assisting the products and serums you are already using to treat various skin issues. The NanoGlobes help to eliminate redness, oxygenate skin, stimulate nerves, reduce pores, and soothe the skin.

### BENEFITS OF TREATMENT:

- Calms sensitive, red, inflamed and irritated skin.
- Creates a more even and brighter skin tone.
- Supports and helps to repair skin's microbiome and barrier function.
- Produces anti-inflammatory and anti-allergy effects to heal skin from the inside out.
- Supports moisture and protection against environmental-damage effects.
- Offers detoxifying and brightening benefits.
- Soothes and cools skin immediately on contact.
- Diminishes the appearance of fine lines and other signs of aging skin
- Helps heal blemishes or other skin imperfections

### PRODUCTS:

- Quiet Wash
- Refine Polish
- Mist Me Spritz
- Hush Hydrate Mask
- Relief Bio-Powder
- Soothe Essence Serum
- Hush Hydrate Gel or Vital Lipid Lotion or Saffron Meristem Cream
- Broad Spectrum SPF 30

### TOOLS:

- Rezenerate WandPro
- Rezenerate Pro Nano Tip
- Rezenerate NanoCleanser Brushes
- Rezenerate NanoGlobes (*frozen or on ice*)

### STEPS FOR TREATMENT

**PRE-CLEANSE:** Apply **Quiet Wash** and emulsify with tepid water. Rinse thoroughly to remove oils, dirt and makeup from skin.

**CLEANSE:** Apply **Quiet Wash** and emulsify with tepid water. Incorporate the **NanoCleanser Brushes** to further help deep clean and detoxify the skin. Rinse thoroughly with a cool towel.

**See page 2 for continued Steps for Treatment, Notes & Contraindications**

**STEPS FOR TREATMENT (CONTINUED)**

**EXFOLIATE:** Apply a dime size amount of **Refine Polish** to face and neck. Gently massage into skin for 3-5 minutes. Remove excess with 4x4s, then with tepid water and sponges to remove any residue.

**TREATMENT:** Use **Soothe Essence** as your treatment Serum, apply liberally all over face with fan brush. Using **HyRez Glide** for slip, use the **WandPro** set to **LOW** with a fresh Tip to perform the infusion portion of the treatment. (See pages 8-10 of the Rezenerate NanoFacial Manual for detailed operating instructions and NanoFacial Sector Maps.)

**MASK:** Apply **Hush Hydrate Mask** with 2-3 shakes of **Relief Bio-Powder** liberally for 5-10 minutes. Using the previously frozen or iced **NanoGlobes**, perform the NanoGlobe Zen Acupressure Massage over the mask. (See page 12 of the Rezenerate NanoFacial Manual for Acupressure Instructions and Map.) Remove residual mask with tepid towel.

**NOURISH:** Wipe skin with wet, soft gauze or cotton soaked in **Mist Me Spritz**.

**HYDRATE:** Apply choice of moisturizer (**Hush Hydrate Gel, Vital Lipid Lotion** or **Saffron Meristem Cream**) to face and neck.

**PROTECT:** Apply **Broad Spectrum SPF 30** to face and neck.

**NOTES AND RECOMMENDATIONS:**

- Prior to using this product, it is recommended to patch test all clients especially with sensitive and Health-Challenged Skin®.
- Wear gloves and sanitize regularly.
- Use steam & hot towels with extreme caution. Have cool towels available if client is agitated by ingredient activity to remove product quickly.
- Do not heavily rub or try to penetrate the product into the skin.
- Check the client's health history for medications, allergies or sensitivities to any ingredients in this product.
- Do not leave product on skin longer than the recommended time.
- Follow all protocols and instructions for the Rezenerate WandPro, Pro Nano Tips, NanoCleansers, and NanoGlobes listed in the Rezenerate NanoFacial Manual. (Manual located in your RAP Online Portal, and Online Education Gateway.)

**CAUTIONS AND CONTRAINDICATIONS:**

- Do not use on skin with open lesions, sores or wounds.
- Do not use on broken/cracked, sun-burned or rashy skin.
- Do not use on open comedones or cystic acne (grades 3 & 4).
- Do not use on clients with compromised barriers or active skin conditions.
- Do not use on clients who have allergies to natural fragrances (Mist Me Spritz.)
- The use of SPF is mandatory for 2 weeks daily following application of this treatment
- Check with physician if client is pregnant, breast feeding, on chemotherapy drugs, have a severe autoimmune disease or any health condition that may need a physician's release to perform this treatment.
- Avoid contact with eyes. Keep out of reach of children.
- Please refer to a Physicians Desk reference manual to assure there will be no conflicts with prescribed or OTC medications and ingredients used in this product.
- If any reactions occur during treatment, cease the use of this product immediately.
- Avoid all contraindications for Rezenerate WandPro, Pro Nano Tips, NanoCleansers, and NanoGlobes listed in the Rezenerate NanoFacial Manual. (Manual located in your RAP Online Portal, and Online Education Gateway.)

\*Please refer to the specific product data sheets with additional questions, concerns and for the complete ingredient listing for each product.

\*Please refer to the Rezenerate NanoFacial Manual for all questions regarding the Rezenerate WandPro, Pro Nano Tips, NanoCleansers, and NanoGlobes. Email Rezenerate clientrelations@rezenerate.com with any questions or concerns.