

# estheticpro product manual



## Thanks for purchasing the EstheticPro Micropen, your new favorite skin care tool!

*Please visit our website, [esthetic-pro.com](http://esthetic-pro.com), to register your EstheticPro Micropen! Once registered, you gain access to our training videos, marketing material, and so much more!*

*We would love to hear how the EstheticPro Micropen transforms your practice! Please share your experience by using #estheticpro and tagging us at @THEestheticpro.*

*Welcome to beauty without compromise!*





## Product Details

1. Body
2. Adapter (220 Volt)
3. Cartridge (not included)

Cartridge Connect

Adjustment of  
Cartridge Length

Power Button and  
Speed Adjustment

Charging Port



*Red: Highest Speed - 18000 rpm*

*Purple: Second Highest Speed - 16000 rpm*

*Blue: Medium Speed - 12000 RPM*

*Cyan: Second Lowest Speed - 10000 rpm*

*Green: Lowest Speed - 8000 rpm*



## How to Operate

- 1 Open the EstheticPro sterile cartridge by peeling the plastic side away from the paper side of the packaging. Remove cartridge.
- 2 Screw in the narrow end of the cartridge into the cartridge connection by turning counter clockwise until hand tight. Do not over tighten.
- 3 Turn on the EstheticPro Micropen by pressing the power button for 2 seconds. The button will flash green, then automatically start on the red setting which is the highest speed.
- 4 Once turned on, adjust the speed by pressing the power button once. The button will change colors. Each of the 5 colors represents a different speed.

- 5 Adjustment of cartridge length should be done after you turn on the EstheticPro Micropen.
- 6 Adjust the cartridge depth by turning the adjustment ring clockwise or counterclockwise. Turning the adjustment ring clockwise will increase the cartridge depth, allowing the cartridge to penetrate deeper. Conversely, turning the adjustment ring counterclockwise will decrease the cartridge depth.
- 7 To turn off, press and hold the power button for 2 seconds.
- 8 Remove cartridge by unscrewing counterclockwise. Dispose of in a sharps container.

## Precautions and Maintenance

- ⊘ Do not use the device on anyone who is sick, has a medical or skin condition or under the age of 18.
- ⊘ Do not treat to the point where blood becomes present.
- ⚠ Only use an AC 220v outlet to charge the device.
- ⊘ Do not repair, disassemble, or alter the device.
- ⊘ Do not touch the device or detach the power cords with wet hands.
- ⊘ Do not use if the charging cord is not in good condition.
- ⊘ When not in use for a long time, unplug it from the outlet.
- ⊘ Cartridges are one use only and disposed of in a sharps container.
- ⊘ Do not allow anyone other than a trained technician to operate the EstheticPro Micropen.
- ⊘ Charge in short bursts and ensure that you're charging it at least once a month to maintain battery life.
- ⊘ When not in use, keep the device stored in a closed container with its power off.
- ⊘ Sanitize after each use: wipe its surface with a disinfectant wipe and then a dry cloth to maintain the integrity of the metal.
- ⊘ Keep the device away from direct sunlight, high temperatures, and humidity - including the use of water and autoclaves.
- ⊘ Inspect and test the device and adapter before each use.
- ⊘ When using the device after it has not been used for an extended period of time, check if it works properly with caution.
- ⊘ The warranty is void if cartridges other than EstheticPro are used. Or you do not follow the operating instructions, precautions or maintain the device.

## Warranty

*If you experience any difficulties with your EstheticPro Micropen, please contact us, we are happy to help. We offer a generous 2-year warranty.*

*Visit our website to register your EstheticPro Micropen and contact us with any questions you may have!*

*[esthetic-pro.com](http://esthetic-pro.com)*

*[info@esthetic-pro.com](mailto:info@esthetic-pro.com)*

